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User's Manual of BabyOno Forehead Thermometer - 117

- Thank you for purchasing our IR Forehead Thermometer. For correct use of this product, be sure to read this User Manual before operation.
- For safe and proper use of this product, be sure to read and fully understand the Safety Precautions contained in this User Manual.
- Keep this Manual with you for convenient reference.
- Keep this Manual somewhere safe and avoid losing it.

Safety precautions

Warning marks and symbols are contained for your safe and proper use of this product and prevention of any injury to you and others. See Table below for description of warning marks and symbols:

Warning

Means a possibility of personal injury in case of improper use.

Notice

Means a possibility of personal injury or property damage in case of improper use.

* Property damage covers any damage to house, family property, domestic animal or pet.

Marks or Symbols

Means Notice with detailed items expressed in words or figures within or beside. The left one means "Do not break".

Means Forbidden with detailed items expressed in words or figures within or beside the mark. The left one means Generally Forbidden.

Means Obligatory with detailed items expressed in words or figures within or beside the mark. The left one means Generally Compulsory.

Means Type-B device.

Warning

It is dangerous for patients to perform self-evaluation and self-treatment based on the measurement results. Be sure to follow doctors' instructions.

- Self-evaluation may cause deterioration of diseases.
Don't touch or blow the infrared sensor.
- A polluted infrared sensor may cause inaccuracy. Clean a polluted infrared sensor with a soft dry cloth in a gentle manner.
- Cleaning with toilet tissue or paper towel may scratch the infrared sensor, causing inaccuracy. Keep the machine out of children's reach.
- Self-measurement by children may cause ear damage. In the case of accidental swallowing of battery or protective film, please consult the doctor at once.
When there is a temperature difference between storage area and measurement site, condition the thermometer for about 30 minutes at room temperature (measurement site).
- Failing to do so may cause inaccuracy.

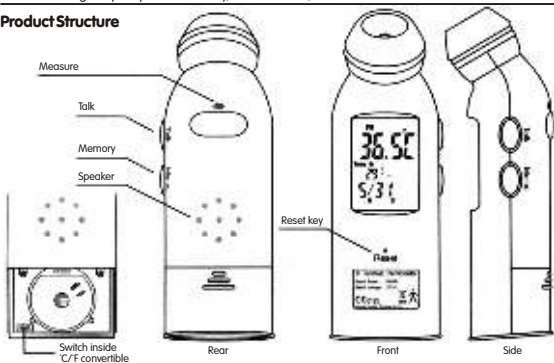
Warning

-  Don't throw batteries into fire.
- The battery is likely to explode in fire.

Recommendations:

- * Don't use the product for any other purpose.
- * It is not a waterproof product; avoid contact with liquids including alcohol and water.
- * Don't leave the product exposed to any chemical solvent, direct sunshine or high temperature.
- * Don't drop, trample or impose any vibration or impact on the product.
- * Don't use a mobile phone nearby when the thermometer is working.
- * The forehead thermometer has been designed for practical use, but is not meant to replace a visit to the doctor.
- * The forehead thermometer is only intended for measuring temperature on the forehead. **DO NOT USE** for measuring body temperature rectally, under the arm, in the mouth or in the ear.

Product Structure

ABC concerning forehead temperature

1. What is forehead temperature?

Forehead temperature is the same temperature as the arterial blood supply under the skin. It is the best determinate of body temperature, and unaffected by the artificial errors and time delays of oral and rectal methods.

2. How to take forehead temperature properly?

* Gently touch the center of the forehead with the probe. Make sure to remove anything covering the area to be measured (hair, hat, wig, and bandages). Otherwise it would insulate the area, resulting in false readings.

3. Why is forehead temperature more accurate than ear temperature?

The reason ear thermometers are considered inaccurate is because the positioning of the probe in the ear canal might affect the accuracy. The forehead thermometer scans the forehead area for temperature given off by the arterial blood supply under the skin, without worry making good position. The gentle scan is comfortable and not objectionable.

Temperature measurements are also affected by the type of measurement method used. Rectal / ear temperature is generally 1/2 to 1 degree higher than oral temperature, and underarm temperature is generally 1/2 to 1 degree lower than oral. The following chart lists general temperature ranges for each method.

Measurement method	Normal temperature range	Fever temperature range
Underarm	95.5 ~ 98.5 °F (35.3 ~ 36.9 °C)	98.9 °F or higher (37.2 °C or higher)
Forehead	95.0 ~ 100.4 °F (35.0 ~ 38.0 °C) NOTE: Measurements for some adults may be lower than 95.0 °F (35.0 °C)	A temperature that is 1.0 ~ 1.5 °F (0.6 ~ 0.8 °C) higher than usual. It is highly recommended that you use our IR Forehead Thermometer to establish the normal temperature range for each person.
Oral	96.6 ~ 99.5 °F (35.9 ~ 37.5 °C)	99.6 °F or higher (37.6 °C or higher)
Rectal / Ear	97.7 ~ 100.3 °F (36.5 ~ 37.9 °C)	100.4 °F or higher (38.0 °C or higher)

How to Take a Measurement:

- 1) Remove protective cap before use and make sure lens is clean. If not, clean with cotton swab dipped in alcohol and let dry.
- 2) Gently position the probe flush on the center of the forehead, midway between the eyebrow and the hairline. Press the button once to start the measurement & you will hear a beep sound. Once the measurement has been completed you will hear a short melody. Then you will hear the result via the speaker (option available only in English) or see it from the LCD.
- 3) The result will be stored in the memory automatically. If 30 sets of memory are stored already, the first set of data will be removed, and others decreased by 1 with the new one at the top. (Note: the recorded data will be removed automatically when it is recorded over 30 days.)
- 4) The maximum number of temperature measurements is 3 times consecutively (due to different measurement places small differences of measurement results are acceptable). It is possible that the correct measurement is obstructed despite the recovery. In order to take further temperature measurements, up to four or more times, wait for 10 minutes before another measure.
- 5) Display of measurement results:
 - A. True value display for result falls between 32.0 ~ 42.9 °C ; 89.6 ~ 109.3 °F
 - B. "HI" displayed for result above 42.9 °C / 109.3 °F
 - C. "LO" displayed for result below 32.0 °C / 89.6 °F
- 6) At completion of measurement, the unit automatically returns to time mode after 30 seconds.
- 7) The display will automatically turn off 3 minutes after the last key was pressed.

Basic functions:

1. Current date and time display: Range of date display falls within Gregorian calendar year 1901~2099, time display mode includes 12 hours shift or 24 hours shift (not user selected). The initial time for a powered system is set at January 1, 2004.
2. Measurement and display of temperature: Measurement ranges from 32.0 °C (89.6 °F) to 42.9 °C (109.3 °F)
3. 30 sets of measurement records stored and searchable: Maximum 30 sets of temperature records stored. Each set has a maximum storage time of 30 days. Temperature is stored together with measuring date, time (morning or afternoon) and status (day time or night time).
4. Voice prompt: report measurement results (in °C or °F) or current time

Advanced operation:

1. Time mode

- System enters time mode when:
 1. Power is on;
 2. No key is pressed for more than one minute.
 - For normal status:
 1. Display time in a sequence of year, month, date, week, hour, minute with a flickering " : " stands for second.
 2. Press "Talk" to report current time.
 3. Press hold "Talk" for 2 seconds to activate or disable the talking function - During measurement, you still hear "didi" no matter whether it is in talking or non-talking mode
 4. Press "Memo" to retrieve the recorded temperature and Press Hold "Memo" for 2 seconds to initiate setup mode.
 5. Press "C / F" to select Celsius or Fahrenheit
 - For setup mode of the hour and date:
 1. Press "Talk" to shift between setting of hour, minute, year, month and date.
 2. Press and hold "Memo" for approximately 3 seconds until you hear the signal, the hour will start flashing, next pressing "Memo" you change one by one the hour. Later press "Talk" and the minutes start flashing. Change the minutes one by one pressing the "Memo" button. If you press the "Talk" button again, you'll enter the year setting mode and you change it by pressing "Memo". In such a way you'll set the next month and day.
 3. Press "Memo" subsequently to resume to normal status.
- * Unit returns to time mode automatically 1 minute after last key pressed.

2. Memory search mode

- Search mode can be activated by pressing "Memo" in normal time mode.
- "- " is displayed if no data are stored;
- Search number is shown for stored data available. Maximum storage is 30 sets of data;
- Press "Memo" to go to the next previously made measurement.
- Unit returns to time mode automatically for 5 seconds after last key pressed.

Recommendations:

1. Measure only the front of the head exposed to the environment. Anything covering the area to be measured (hair, hat, wig, bandages) would insulate the area, resulting in falsely high readings.
2. Wait about 30 seconds before measuring the same patient again to avoid excessive cooling of the skin.
3. When exposed to changing room temperature, the device should at least remain in the same room in which the temperature will be measured for 15 minutes prior to the measurement
4. Always cover the thermometer with the protective cap supplied.

Maintenance, storage and calibration:

Maintenance:

1. Remove any stains on the body with a soft and dry cloth.
2. Don't wash with water or detergent containing abrasive or benzene.

Storage:

1. Always put the protective cap over the probe if the unit is not in use.
2. Don't leave the thermometer exposed to or at high risk of direct sunshine, high temperature, damp, fire, flame, vibration or impact.
3. Remove the battery in the case of long time without use.

CALIBRATION :

This thermometer is calibrated at the time of manufacture. If the thermometer is used according to the instructions, periodic re-calibration is not required. If at any time you question the accuracy of the measurement, please contact the retailer immediately

- * Don't attempt to modify or reassemble the thermometer.

TROUBLES and TROUBLESHOOTING:

Troubles	Checklists	Countermeasures
No response	1) Battery used up? Or 2) Battery in wrong polarity?	1) Change new battery, or 2) Adjust battery to proper polarity.
Battery symbol on LCD	Low battery	Change new battery
"Lo" on screen (result below 32.0 °C / 89.6 °F).	Thermometer correctly pointed at the forehead?	Follow User's Manual to point at the forehead.
"Hi" on screen (result above 42.9 °C, 109.3 °F).	Dirty lens?	Clean the lens
"ErrE" on screen	Thermometer temperature beyond the scope of 10.0 ~ 40.0 °C, (50.0 °F ~ 104.0 °F)	Condition the thermometer to room temperature (10.0 ~ 40 °C, 50.0 ~ 104 °F) before measuring.
"ErrU" on screen	Don't measure after pressing Measure.	Follow steps in User's Manual for proper measurement.
"ErrH" on screen	Errors detected during self-check.	Consult a repairman.

Replacing the battery:

Remove the battery compartment door. Use only a new battery (CR2032). Replace the battery compartment tight.

**SPECIFICATIONS:**

- Product Name: IR Forehead Thermometer
- Power supply: 3 Volt (CR2032)
- Power consumption: <0.05W (for voice prompt)
- Temperature sensing part: IR sensor
- Measuring range: 32.0 °C ~ 42.9 °C (89.6 °F to 109.3 °F)
+ _0.2 °C (or +/ - 0.4 °F)
- Measuring accuracy: from 36.0 °C ~ 39.0 °C (or 96.7 °F to 102.2 °F)
+ _0.3 °C (+/- 0.5 °F)
from 32.0 °C ~ 35.9 °C (or 89.6 °F to 96.6 °F)
from 39.1 °C ~ 42.9 °C (or 102.3 °F to 109.3 °F)
- Outer dimension: 110(L) x 27(W) x 35(H) mm
- Weight: 63g
- Working environment:
 - Temperature: 10 ~ 40.0 °C (50.0 °F to 104.0 °F)
 - Relative humidity: 30%RH ~ 80%RH
- Storage environment:
 - Temperature: -20.0 ~ 40.0 °C (-4.0 °F to 104.0 °F)
 - Relative humidity: 30%RH ~ 85%RH

11. EQUIPMENT not suitable for use in the presence of a FLAMMABLE ANAESTHETIC MIXTURE WITH AIR or WITH OXYGEN OR NITROUS OXIDE.

12. CONTINUOUS OPERATION;

*Dispose of used batteries with care; consult your distribution center for details.

ACCESSORIES:

Protective cap: 1 pc; User Manual: 1 pc; Probe Covers: 3 pcs;

WARRANTY:

One-year warranty is available from purchasing date, excluding user-caused failures listed below:

(1) Failure resulted in unauthorized disassembly and modification.

(2) Failure resulted in unexpected drop during application or transportation.

(3) Failure resulted in operation not in accordance with proper instructions in User Manual.

Present the Warranty Card with the seal of the distribution center (include name and address) for free repair. Consult the sales office for free service.

NOTES:

1) In an environment with electrostatic discharge, the unit may malfunction and may require the user to reset the unit.

2) Dispose with care; please consult the retailer for details.

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